

Moisture Wicking Fabric

with Tri-Go™ Technology

Reduces moisture

Captures and transports moisture away from the skin


Reduces bacteria

Polyester fabric acts as a lubricant, reducing skin on skin friction

Reduces odour

Device reduces colonisation of bacteria and yeasts within fabric

Soft and conformable fabric designed to help manage the conditions associated with Intertrigo

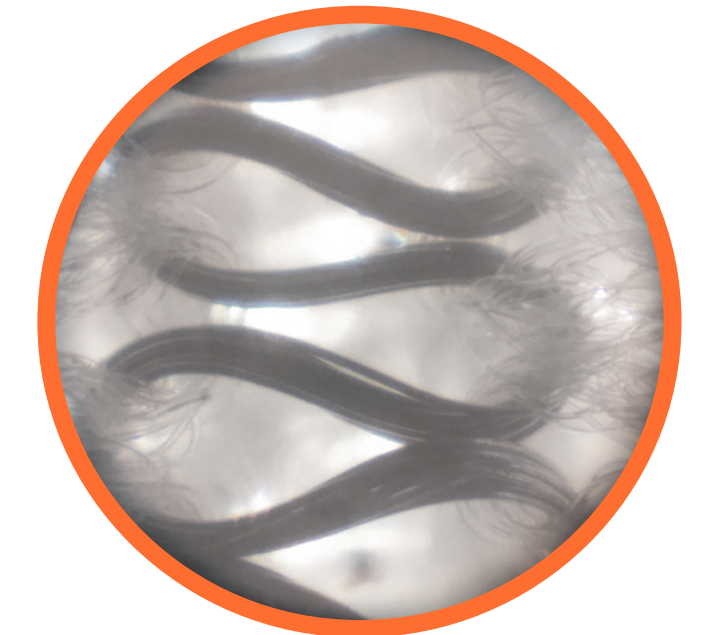

Moisture Wicking Fabric with Tri-Go™ Technology is a stretch polyester fibre impregnated with antimicrobial metallic silver and is a non-sterile skin protectant. The device fabric is designed to provide moisture transportation to keep skin dry while the antimicrobial silver within the fabric reduces odour.

The polyester fabric acts as a lubrication aid thereby reducing skin-to-skin friction. Provides a protective environment for the skin and reduces colonisation of bacteria and yeasts such as Staphylococcus aureus, Staphylococcus epidermidis, Pseudomonas aeruginosa and Candida albicans within the fabric.

Moisture Wicking Fabric

with Tri-Go™ Technology

Fabric is a skin protectant

Polyester acts as a lubricant reducing skin on skin friction

Device provides moisture transportation

Enables the skin to be kept dry by creating a pathway to remove sweat from the body

Antimicrobial silver within the dressing prevents odour

Enables the reduction and colonisation of bacteria and yeasts such as Staphylococcus aureus, Staphylococcus epidermidis, Pseudomonas aeruginosa and Candida albicans within the device.

Can remain in situ for up to 5 days

The dressing also conforms to anatomical contours and can be cut to size depending on patient requirements

Device can be made available without prescription


Product selection

| Product Selection | | |
|-------------------|------------|-------|
| 25cm x 90cm | 10" x 36" | Pouch |
| 25cm x 365cm | 10" x 144" | Roll |


Indications

Moisture wicking fabric with Silver (Ag) is indicated for use in the management of skin-on-skin friction.

Application Guide

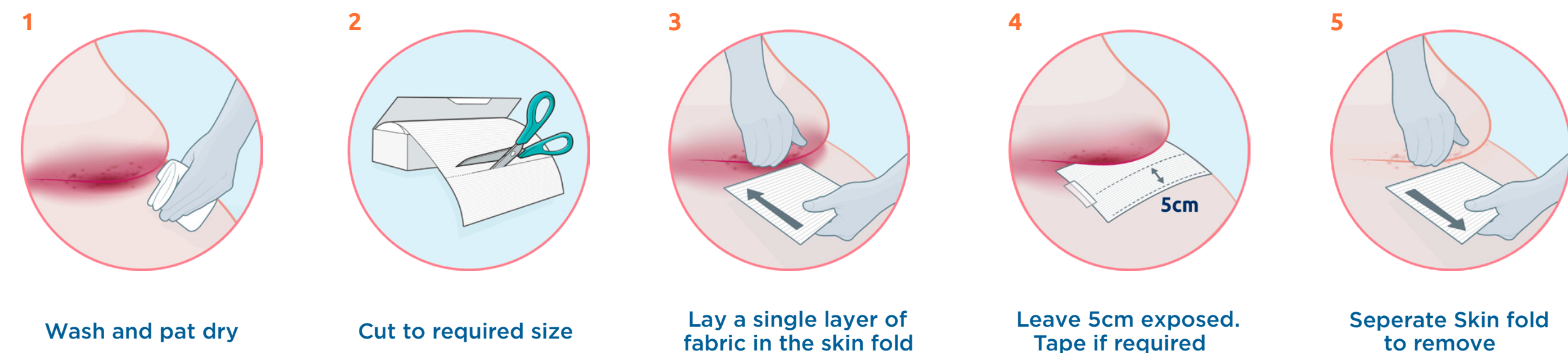

Possible areas affected

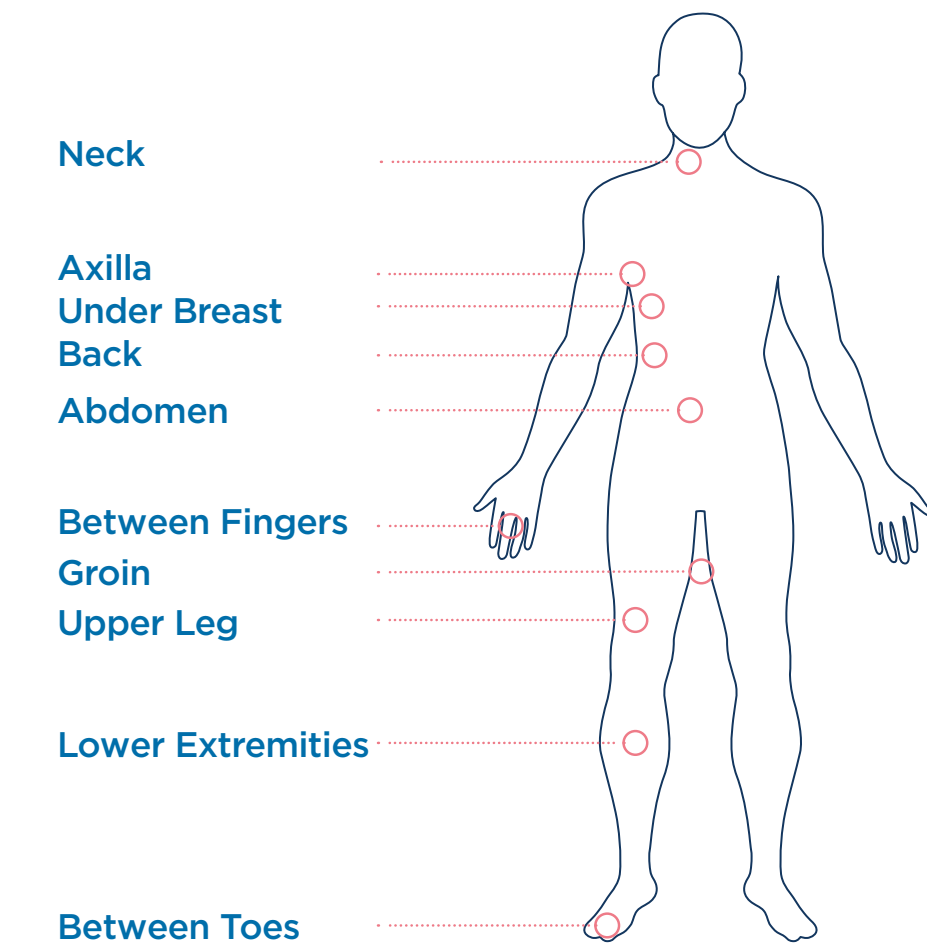

Contact

Advanced Medical Solutions Ltd
Premier Park
33 Road One
Winsford Industrial Estate Winsford
Cheshire CW7 3RT UK

Tel: +44(0) 1606 863 500
Email: sales@admedsol.com
www.admedsol.com



Helping you help others

Advanced Medical Solutions excels in the research and development of medical devices, through continuous product development and by working closely within the market place we meet the needs of the clinician, patient and our commercial partners.