



# Patient Pathway Guidance

The patient's wound has been closed using LiquiBand® XL, a sterile skin closure system that uses a combination of liquid adhesive and mesh.



**LiquiBand® XL is designed to naturally slough off or it can be removed by a Health care provider following the removal instructions identified in the Instructions For Use.**

**LiquiBand® XL provides effective wound closure for easily approximated skin edges of wounds with the added benefit of a microbial barrier.**

- ✓ *LiquiBand® XL should remain in place until the wound/incision has healed (typically 7 to 14 days).*
- ✓ *During this time, LiquiBand® XL should be kept dry. If directed by the health care practitioner, the wound may be briefly wet in a shower if dried immediately thereafter by gently blotting with a soft towel.*
- ✓ *The wound should not be soaked or scrubbed. Patients should not swim or otherwise immerse the wound in water for prolonged periods.*
- ✓ *If a protective wound dressing is being used over LiquiBand® XL, it should be replaced with a dry dressing after showering or bathing.*
- ✓ *The patient should be instructed not to scratch, rub or pick at LiquiBand® XL.*
- ✓ *The patient should be advised not to apply topical ointments, lotions or liquids to the wound while LiquiBand® XL is in place, as this may cause it to peel away before the wound has fully healed.*
- ✓ *The patient should be instructed not to engage in strenuous physical activity that may cause tension on the wound or cause perspiration to wet LiquiBand® XL.*
- ✓ *Report any discomfort or other concerns regarding your wound to your doctor.*

**LIQUIBAND® XL**  
SKIN CLOSURE SYSTEM

